RELIANCE ORTHODONTIC PRODUCTS, INC.
1540 W. Thorndale Avenue,
Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

 RelianceOrthodontics.com

 ITC TRANSLATIONS
ITC USA
2000 PGA Blvd., Suite 4440
Palm Beach Gardens, FL 33408

 EC REP MD

EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

AUSTRALIAN SPONSOR
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney,
NSW 2000, Australia

U.K. PERSON RESPONSIBLE
EMERGO CONSULTING (UK) LIMITED
c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge
CB24 9BZ
England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE
MedEnvoy Global BV
Leidschendam-Voorburg, Zug Branch Office
Gotthardstrasse 28, 6302 Zug, Switzerland

ASSURE®
UNIVERSAL
BONDING RESIN

INSTRUCTIONS FOR USE

 V3 OCT.2023
1639

 **RELIANCE®**
ORTHODONTIC
PRODUCTS

SCAN FOR MORE INFORMATION

EN - ENGLISH

INDICATIONS FOR USE: Assure® Universal Bonding Resin is intended to increase adhesion to wet or dry, normal and atypical enamel surfaces including: hypocalcified, fluorosed, deciduous enamel and dentin. It will bond to roughened metal (amalgam, gold or stainless steel) and composite restorations without additional primers.

DIRECTIONS FOR USE:

ENAMEL PREPARATION: Prophy the enamel surfaces with 1st & Final® pumice. Rinse and dry thoroughly. Etch the enamel surface for 30 seconds. Rinse and dry thoroughly. If using a self etching scrub the SEP into the enamel for 5 seconds, air dry thoroughly.

APPLICATION OF BONDING RESIN: Apply one coat of Assure® Universal Bonding Resin to each prepared tooth and stroke over to ensure penetration. Lightly air dry. Assure can be light cured to eliminate bracket drift, but not necessary.

Proceed with the application of any chemical or light cure paste with bracket.

BONDING TO COMPOSITE, GOLD, AMALGAM, STAINLESS STEEL & PLASTIC: Prophy, rinse and dry. Sandblast, rinse and dry. Apply one coat of Assure® Universal Bonding Resin and lightly dry. Proceed with the application of paste and bracket.

BONDING TO PORCELAIN: Prophy, rinse and dry. Sandblast, rinse and dry. Apply Porcelain Etch, wait 4 minutes. Rinse and dry. Apply one coat of Porcelain Conditioner, wait 2 minutes. Dry for 30 seconds. Apply one coat of Assure, air dry and light cure for 10 seconds. Proceed with bracket placement.

SPECIAL SAFETY NOTICE: Brush applicators are single use devices. It is required that a new brush applicator is used for each application.

PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spinal area immediately with copious amounts of water for several minutes.

Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

Protective eyewear should be worn by the patient and dental staff

Flammable.

TEMPERATURE LIMITATION: Store at room temperature (15-25°C).

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINdications: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: Federal law restricts this device to sale or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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ДИРЕКТИВА ЗА УПОТРЕБА: Prophy the enamel surfaces with 1st & Final® pumice. Rinse and dry thoroughly. Etch the enamel surface for 30 seconds. Rinse and dry thoroughly. If using a self etching scrub the SEP into the enamel for 5 seconds, air dry thoroughly.

ПРИЛОЖЕНИЕ: Apply one coat of Assure® Universal Bonding Resin to each prepared tooth and stroke over to ensure penetration. Lightly air dry. Assure can be light cured to eliminate bracket drift, but not necessary.

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ПРИЛОЖЕНИЕ: Prophy, rinse and dry. Sandblast, rinse and dry. Apply one coat of Assure® Universal Bonding Resin and lightly dry. Proceed with the application of paste and bracket.

ПРИЛОЖЕНИЕ: Prophy, rinse and dry. Sandblast, rinse and dry. Apply Porcelain Etch, wait 4 minutes. Rinse and dry. Apply one coat of Porcelain Conditioner, wait 2 minutes. Dry for 30 seconds. Apply one coat of Assure, air dry and light cure for 10 seconds. Proceed with bracket placement.

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Директива за употреба: Prophy the enamel surfaces with 1st & Final® pumice. Rinse and dry thoroughly. Etch the enamel surface for 30 seconds. Rinse and dry thoroughly. If using a self etching scrub the SEP into the enamel for 5 seconds, air dry thoroughly.

Приложение: Apply one coat of Assure® Universal Bonding Resin to each prepared tooth and stroke over to ensure penetration. Lightly air dry. Assure can be light cured to eliminate bracket drift, but not necessary.

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Приложение: Prophy, rinse and dry. Sandblast, rinse and dry. Apply one coat of Assure® Universal Bonding Resin and lightly dry. Proceed with the application of paste and bracket.

Приложение: Prophy, rinse and dry. Sandblast, rinse and dry. Apply Porcelain Etch, wait 4 minutes. Rinse and dry. Apply one coat of Porcelain Conditioner, wait 2 minutes. Dry for 30 seconds. Apply one coat of Assure, air dry and light cure for 10 seconds. Proceed with bracket placement.

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