RELIANCE ORTHODONTIC PRODUCTS, INC.
1540 W. Thorndale Avenue,
Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

 RelianceOrthodontics.com

 ITC TRANSLATIONS
ITC USA
2000 PGA Blvd., Suite 4440
Palm Beach Gardens, FL 33408

 EC REP MD

EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

AUSTRALIAN SPONSOR
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney,
NSW 2000, Australia

U.K. PERSON RESPONSIBLE
EMERGO CONSULTING (UK) LIMITED
c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge
CB24 9BZ
England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE
MedEnvoy Global BV
Leidschendam-Voorburg, Zug Branch Office
Gotthardstrasse 28, 6302 Zug, Switzerland

CROWN LOK™ LUTING CEMENT CONTAINS FLUORIDE

INSTRUCTIONS FOR USE

 V3 OCT.2023
1639

 **RELIANCE®**
ORTHODONTIC
PRODUCTS

SCAN FOR MORE INFORMATION

EN - ENGLISH

INTENDED USE: Crown Lok™ is intended for use as a cement for metal crowns and orthodontic bands.

INSTRUCTIONS FOR USE:

ENAMEL PREPARATION

Proply the surface with 1st & Final® oil-free pumice. Rinse and dry. Apply the etching agent to the tooth. Allow 30 seconds for etching. Rinse and dry thoroughly. With a microetcher, sandblast the inside of the crown, band or bridge.

BONDING & CEMENTATION: Mix equal amounts of Crown Lok™ Paste A and Paste B for 15 seconds on the supplied mixing pad and place inside the appliance. The working time from start of mix is 2 minutes. Final set will occur 5 minutes from start of mix.

FOR SELF MIXING DUAL-SYRINGE DELIVERY SYSTEM

1. Remove cap from the SELF MIXING SYRINGE
2. Place mixing tip by lining up notch in tip hub (A) with the syringe notch (B) and rotating in a clockwise direction until it is in a locked position.

3. Syringe material into band/crown or appliance using mixing tip. **Do not re-use the tips.** The SELF MIXING SYRINGE mixing tip should be removed and discarded after each patient use. If used intraorally, the syringe should be cold sterilized prior to storage.

4. Replace the cap on the syringe after use.

SPECIAL SAFETY NOTE CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES. APPLY DISPOSABLE BARRIER SLEEVES/WRAPS BEFORE EACH PATIENT

PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

Protective eyewear should be worn by the patient and dental staff.

TEMPERATURE LIMITATION: Store refrigerated (2-8°C)

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous-waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: Federal law restricts this device to sale or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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