RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue,
Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA
2000 PGA Blvd., Suite 4440
Palm Beach Gardens, FL 33408

EC REP MD

EMERGO EUROPE
Westervoortseindijk 60
6827 AT Arnhem
The Netherlands

AUSTRALIAN SPONSOR
EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE
EMERGO CONSULTING (UK) LIMITED
c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge
CB24 9BZ
England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV
Leidschendam-Voorburg, Zug Branch Office
Gotthardstrasse 28, 6302 Zug, Switzerland

RELIANCE S.E.P.TM SELF ETCHING PRIMER

INSTRUCTIONS FOR USE

V3 OCT.2023

CE
1639
RELIANCE[®]
ORTHODONTIC
PRODUCTS

SCAN FOR MORE INFORMATION

EN - ENGLISH

INDICATIONS FOR USE: The Reliance S.E.P.TM is a self etching primer intended to be used for the preparation of an enamel surface prior to bonding with a light cure orthodontic adhesive.

INSTRUCTIONS FOR USE:

TO DISPENSE PRODUCT FOR USE:

- Pray, rinse and dry the enamel.
- Position both tips over the light impervious mixing well and push (Button 1 below) one with a quick, forced motion. Initial dispensing of S.E.P.TM may require one or two pushes to start the flow (Note: Refill must reach room temperature in order to dispense properly). The ratio that is dispensed is 3:1 (yellow/clear) liquids with a Micro Brush[®].
- Mix the dispensed liquids thoroughly.
- Saturate a Micro Brush[®] and vigorously scrub the enamel surface for 5 seconds where the bracket will be placed and slightly beyond. Slide the shield over the mixed solution to protect from ambient light. This mixture can be used for up to 6 hours if protected from light with the shield.
- Dry the enamel surface thoroughly.
- Optional – Apply 1 coat of Assure[®] Plus and air dry to significantly increase strength
- Apply bracket with light cure paste and light cure.
- Replace cap tightly and leave the cartridge in the dispenser.

TO LOAD A NEW CARTRIDGE:

- Allow Refill Cartridge to reach room temperature prior to use.
- SLIDE EJECT BUTTON SEVERAL TIMES TO RETRACT INTERNAL PLUNGERS BEFORE INSERTING NEW CARTRIDGE. (Refer to Figure 2).
- With Reliance logos facing up on both cartridge and dispenser, line up the arrow on the dispenser and the arrow on the cartridge then insert cartridge into the S.E.P.TM dispenser holding small barrel toward the top and pushing the Reliance logos towards each other to snap into place.
- Remove orange cap.

TO EJECT THE CARTRIDGE

- To remove cartridge, turn the dispenser so the Reliance logo is facing down. With cap removed, slide the eject button.
- Pull the cartridge out of the dispenser.
- Discard cap and cartridge.
- SLIDE EJECT BUTTON SEVERAL TIMES TO RETRACT INTERNAL PLUNGERS BEFORE INSERTING NEW CARTRIDGE.

- CAUTION:** Irritant. Avoid contact with eyes or eyes. If accidentally instilled within the eye, flush with copious amounts of water for a minimum of 15 minutes and seek medical attention if irritation persists. If contact occurs with skin or other tissues, wash area with soap and water immediately. Seek medical attention if irritation persists.
- Flammable

- PRECAUTIONS:**
1. Avoid applying excessive amounts of the S.E.P.TM to the enamel surface as the acid contained in the S.E.P.TM can irritate the gingiva (soft tissue) as well as potentially reduce bond strength.

2. Dropping the dispenser will shorten its anticipated life. Dropping with cartridges inserted can also do underetermined damage to the tips. THE ANTICIPATED LIFE OF THE S.E.P.TM DISPENSER IS ONE YEAR.

3. MicroBrush applicators are single use devices. It is required that a new brush applicator is used for each application.

4. Removing a partially used cartridge and re-inserting it does cause added stress on the dispenser and cartridge. The cartridge is not intended to be removed until completely used, then to be removed and discarded. If necessary, the cartridge can be removed when only partially used and reinserted for continued use. However, it is important to know that resetting and removing a partially used cartridge requires continued clicking to get the dispensing tip back into alignment with the dispensing slot.

5. The dispenser should never be placed in an autoclave or fully submerged in any type of cleaning solution. It is best to disinfect the dispenser by wiping with a glutaraldehyde solution or denatured alcohol. Please note that certain surface cleaning agents that contain, but not limited to, substances such as Chlorides, Quaternary Compounds and Citric Acid should be avoided as they may damage dispenser.

6. Do not expose the cartridge or dispenser to areas of high light exposure for any extended time period or the material may cure in the cartridge. The orange tint of the cartridge helps to reduce light penetration to the light sensitive material inside, however it does not eliminate it completely. Caps should also be replaced on the tips of the cartridges immediately after dispensing since the opening at the tip allows unprotected resin to be exposed to ambient light.

7. Mixing wells should be sterilized in a cold sterilization unit, DO NOT HEAT STERILIZE. DO NOT REFURGERATE DISPENSER.

- TEMPERATURE LIMITATIONS:** Store extra refill cartridges refrigerated (2-8°C). Do not freeze. Allow product to reach room temperature prior to use.

- Protective eyewear should be worn by the patient and dental staff.

- PATIENT POPULATION:** The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

- CONTRAINDICATIONS:** Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

- SAFE DISPOSAL:** Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulations. Do not reuse waste disposal contractor or collection site except for clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

- WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Reliance warrants the S.E.P.TM Dispenser for 6 months from the date of purchase with limitations that include misuse or abuse.

- RX ONLY:** U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

- NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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