**RELIANCE ORTHODONTIC PRODUCTS, INC.**  
1540 W. Thorndale Avenue,  
Itasca, IL 60143 USA  
1-800-323-4348 Made in USA | Rx Only  
RelianceOrthodontics.com

 RelianceOrthodontics.com

 ITC TRANSLATIONS  
ITC USA  
2000 PGA Blvd., Suite 4440  
Palm Beach Gardens, FL 33408

EC REP MD

**EMERGO EUROPE**  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

**AUSTRALIAN SPONSOR**  
**EMERGO AUSTRALIA**  
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney,  
NSW 2000, Australia

**U.K. PERSON RESPONSIBLE**  
**EMERGO CONSULTING (UK) LIMITED**  
c/o Cr360 – UL International  
Compass House, Vision Park Histon  
Cambridge  
CB24 9BZ  
England, United Kingdom

**SWITZERLAND AUTHORIZED REPRESENTATIVE**  
MedEnvoy Global BV  
Leidschendam-Voorburg, Zug Branch Office  
Gotthardstrasse 28, 6302 Zug, Switzerland

**ULTRA BAND-LOK™**

## INSTRUCTIONS FOR USE

 V3 OCT.2023  
1639

 **RELIANCE®**  
ORTHODONTIC  
PRODUCTS

SCAN FOR MORE INFORMATION

### EN - ENGLISH

#### INDICATIONS FOR USE:

Ultra Band-Lok® is intended for use as an orthodontic band cement and for occlusal buildups.

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#### If using the Luer-Loc syringes:

- Unscrew and remove the black storage cap from the tip of the syringe.
- Put the tip in place and twist clockwise until the tip is locked in position.
- When the procedure is completed, remove the tip and replace the black storage cap to keep the Ultra Band-Lok® protected from light.

**OCLUSAL BUILDUP APPLICATION:** Prophy, rinse and dry. Etch occlusal surface for 30 seconds, then rinse, dry and isolate. Apply Ultra Band in the amount and shape required to the occlusal surface. Light cure for 20 seconds. Note: a self-etching primer can be utilized in place of phosphoric acid etching.

**Important:** Ultra Band-Lok® is sensitive to ambient light. Shield exposed unpolymerized product until the product is ready to be light cured.

**PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

**Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.**

**TEMPERATURE LIMITATION:** Fluoride version store refrigerated (2–8°C). Light Sensitive. Avoid excessive heat. Allow product to reach room temperature prior to use.

**NonFluoride version room temperature storage (15–25°C).**

**Light Sensitive.**

Protective eyewear should be worn by the patient and dental staff

**PATIENT POPULATION:** The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

**CONTRAINdications:** Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

**SAFE DISPOSAL:** Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous-waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

**WARRANTY:** Reliance Orthodontic Products Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**RX ONLY:** Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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