



A2™

V4 FEB.2020

Aligner Attachments

INDICATIONS FOR USE: A2™ Aligner Attachments are intended for use in the creation of thermoplastic aligner attachments.

PRODUCT FEATURES:

The Oval Attachment Form can be used for the retention of thermoplastic aligners and for horizontal or vertical placement on the tooth depending on the height of the crown.

The Rectangular Attachment form can be used for individual tooth movement and for extrusion of teeth.

INSTRUCTIONS FOR USE:

Enamel Preparation:

- 1) Prophylaxis, rinse and dry the enamel
- 2) Acid etch with phosphoric acid for 30 seconds, rinse and dry thoroughly (or) condition with a self etching primer and dry.
- 3) Apply a coat of Assure Plus®, lightly dry with air.

Attachment Mold Preparation:

- 1) Insert Attachment form into the handle provided. Using L.C.R™ or FlowTain™, slightly under fill the Attachment mold with the L.C.R.™
- 2) Firmly place the Attachment mold against the tooth to ensure proper adaptation,
- 3) Light cure for 10-20 seconds depending on the light intensity. Remove form after curing Note: Excess pressure will result in unwanted composite flash. The oval Attachment form has small vertical aligners to assist in correct vertical, horizontal and angular placement.



Note: A2 Aligner attachments are single use devices

Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

TO REORDER:

Ref# AR for Rectangular Aligner Attachment Forms, Ref# AO for Oval Aligner Attachment Forms
Ref# BH for 1 x Attachment Form Handle. Also Available: Ref# ABUT for 1 x Button Hook Attachment


WARRANTY: Reliance Orthodontic Products Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Rx Only: Federal law restricts this device to sale by or on the order of a dental professional.



Reliance Orthodontic Products, Inc.

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com

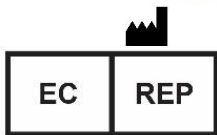

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Australian Sponsor:
Emergo Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia



Check out our new learning
platform Reliance U!
www.RelianceU.tech

Made in the USA

Scan for more information