<span><span>🇬🇧</span></span> ENGLISH
<b>INDICATIONS FOR USE:</b> The ASK® All Surface Kit is intended for use to increase adhesion to normal, enamel, amalgam, gold, stainless steel, porcelain, zirconia, composite restorations and acrylic temporary/periotic teeth.
<b>DIRECTIONS FOR USE:</b> <b>EMAL PREPARATION</b> Prophy the surface to be bonded with 1st & Fina® oil-free Pumice. Rinse and dry. Apply the etching agent to the tooth. Allow 30 seconds for etching. Rinse and dry thoroughly. <b>NOTE:</b> A self etching primer may also be used in place of phosphoric acid etchant. If using a self etching primer, proceed to application of bonding resin.
<b>Application of Assure® Plus with Light Cure Paste:</b> Apply one coat of Assure® Plus to all prepared teeth. Lightly dry with air. Light cure for 10 seconds. Proceed with placement of bracket.
<b>Application of Assure® Plus with Chemical Cure Paste:</b> Apply one coat of Assure® Plus to all prepared teeth. Lightly dry with air. Light cure for 10 seconds. Proceed with placement of bracket.
<b>BONDING TO COMPOSITE, GOLD, AMALGAM, STAINLESS STEEL OR ACRYLIC TEMPORARIES:</b> Prophy, rinse and dry. Sandblast, rinse and dry. Apply one coat of Assure® Plus to all prepared teeth. Lightly dry and lightly dry. Light cure the Assure Plus first for 10 seconds. Proceed with placement of bracket.
<b>BONDING TO PORCELAIN AND ZIRCONIA:</b> Prophy, rinse and dry. Sandblast, rinse and dry. Apply one coat of Porcelain Conditioner and allow to remain on surface for 2 minutes. Thoroughly air dry. Apply one coat of Assure® Plus and lightly dry with air. Light cure for 10 seconds. Proceed with placement of bracket.
<b>BONDING TO DENTIN:</b> Prophy, rinse and dry the surface to be bonded. Acid etch for 30 seconds then rinse and dry, but DO NOT DESICcate – LEAVE MOIST. Apply one coat of Assure® Plus and lightly dry. Light cure for 10 seconds. Proceed with application of paste and bracket.
❖ Special Safety Note: Brush applicators are single use devices. It is required that a new brush applicator is used for each application. ❖ Etchmaster tips are single use devices. Dispose of Etchmaster tips after each use. A new Etchmaster tip is required for each patient.
Note: Avoid evaporation, replace cap between uses.

**SAFETY GUIDELINES FOR THE ETCHMASTER** – Read the instructions and SDS carefully before using this device. The manufacturer, distributor, or retailer of this product can exercise no control over the use of the EtchMaster®. DO NOT EXCEED 45 PSI

❖ This is a disposable device. Dispose after single patient use regardless of the remaining quantity of powder in tip. DO NOT attempt to sterilize, refill or reuse the tips.

Noting and pressurizing the device more than once will degrade device operation. Rubber dam must be used to protect adjacent teeth and soft tissue. Matrix band should be used in place of porcelain conditioner and glass slides before actual use.

Due to the alumina powder, it is necessary to protect the eyes of anyone present in the treatment room during the procedure, namely the patient, assistant and dentist. For patient protection goggles and/or face (eyes and nose) covering with powder resistant material should be used. All wearing should always be used to provide additional eye protection. We recommend dental personnel wear face shields over safety glasses and masks for extra protection.

Do not use oxygen or other flammable or toxic gases as propellant. Excessive air abrasing will erode most surfaces. Practice and experiment on typodonts, penies, stainless steel, extracted teeth, or glass slides before actual use. These surfaces will simulate both precious and non-precious alloys and porcelain.

**REUSE INSTRUCTIONS:** Etchmaster handpiece can be heat sterilized between uses. **CAUTION:** Do not point the EtchMaster® nozzle towards your face or eyes. Always wear protective safety glasses when using this device. NEVER refill and/or reuse the device once it has run out of material, safely dispose after use.

❖ Flammable  
❖ PRECAUTIONS: Unpolymerized resins may cause skin sensitization in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

Protective eyewear should be worn by the patient and dental staff.  
❖ **TEMPERATURE LIMITATION:** Store at room temperature (15-25°C). **IMPORTANT – REVIEW SAFETY PRECAUTIONS PRIOR TO USE. MANUFACTURER IS NOT RESPONSIBLE FOR DAMAGES DUE TO MISUSE OF THE PRODUCT.**

<b>STEP 1</b> – Check tip color code mark to identify abrasive grade
<b>STEP 2</b> – Hold tube end of the EtchMaster® tip with one hand and the handheel connector with the other and firmly insert tube end into the connector plastic collet.
<b>STEP 3</b> – Pull needle forward until stop to unseat abrasive material.
Note – Turning nozzle prior to pulling reduces required pulling strength.
<b>STEP 4</b> – Direct nozzle at target surface and apply air flow
<b>STEP 5</b> – Remove air abrasion tip from connector by pulling back on plastic collet and then pulling device away from the connector.
<b>PATIENT POPULATION:</b> The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by orthodontic specialist.
<b>CONTRAINDICATIONS:</b> Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.
<b>SAFETY DISPOSAL:</b> Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous waste disposal contractor or collection site except for empty containers. Do not place disposed of as non-hazardous waste, as a hazardous or special waste collection point.
<b>WARRANTY:</b> Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not assume liability for replacement or correction of defective products resulting from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.
<b>RX ONLY:</b> Federal law restricts this device to sale to or on the order of a dental professional.
<b>NOTICE:</b> Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.
<b>TO REORDER:</b> #EMHAR, #PLUS, #MO, #EMTLS, #EMTRS

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<b>SWITZERLAND AUTHORIZED REPRESENTATIVE</b> Leidenvoy Global BV Meidschenod-Voorburg, Zug Branch Office Gotthardstrasse 28, 6302 Zug, Switzerland
---

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