RELiance ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

	EC REP	MD

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gothardstrasse 28, 6302 Zug, Switzerland

BAND-LOK®

INSTRUCTIONS FOR USE

1639

V3 OCT.2023

RELiance®

ORTHODONTIC PRODUCTS

SCAN FOR MORE INFORMATION

EN - ENGLISH
INTENDED USE: Band-Lok® is a two paste, dual-cure adhesive intended for use as a band and appliance cement.
INSTRUCTIONS FOR USE:
Step 1. Prophly, rinse, dry and isolate tooth to be banded. Acid etching of the enamel is not required for cementing bands, however, it will increase strength required for high stress appliances.
Step 2. On a mixing pad place equal parts of Band-Lok® Pastes A and B (A 1 inch strip of each part will provide enough cement for 4 bands). If paste is not going to be mixed immediately, shield from light.
Step 3. Mix Pastes A and B thoroughly for 10 seconds and place in band. Seat band and clean off excess flash.
Step 4. At this point you have three curing options:

- Light cure the cement from occlusal for 10 seconds. Cement can be exposed to saliva at this point. Final cure will occur in 5 minutes.

- Light cure the cement from the occlusal for 30 seconds. Cement is now completely cured and can be exposed to forces immediately.

- Allow the cement to chemically cure on its own; complete polymerization will occur in 10 minutes.

IMPORTANT: Band-Lok® will adhere to metal and stainless-steel instruments! Before using, spray instruments with a light coat of PAM® or oil-free silicone before use.

SELF MIXING DUAL-SYRINGE DELIVERY SYSTEM

- Remove cap and place mixing tip on syringe
- Syringe material into band/crown or appliance using mixing tip.
- Replace the cap on the syringe after use.

⚠️ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. Wash hands and other exposed areas with mild soap and water before eating, drinking or smoking and when leaving work. Provide good ventilation in process area to prevent formation of vapor. Rinse eyes immediately with plenty of water. Obtain medical attention if pain, blinking or redness persists.

CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES. APPLY DISPOSABLE BARRIER SLEEVES/WRAPS BEFORE EACH PATIENT

⚠️ **Important! Self-Mixing Tips are single use devices. A new tip is required for each patient.**

⚠️ Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

⚠️ **TEMPERATURE LIMITATION:** Store refrigerated (2-8 Degrees C.)

Protective eyewear should be worn by the patient and dental staff.

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk.

RX ONLY: Federal law restricts this device to sale by or on the order of a dental professional

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.
TO REORDER: #BL, #BLB, #BL SA, #BLSA, #BLSB, #SMBL, #SMBLB

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