RELiance ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

EMERGO AUSTRALIA

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower 1l, Darling Park, 201 Sussex St, Sydney,

NSW 2000, Australia

U.K PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

England, United Kingdom

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
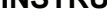
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
SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

EN - ENGLISH

INDICATIONS FOR USE: Bond Aligner® is intended for use as an orthodontic bonding adhesive for brackets, appliances and button formation to thermoplastic aligner surfaces. It is not indicated for use with cloudy (Class C) thermoforming materials.

INSTRUCTIONS FOR USE:

1.Apply Bond Aligner® clear aligner adhesive to the appliance to be bonded. Aligner should be clean and free of any oral fluids or biofilm where the attachment will be bonded.

2.Seat appliance on thermoplastic aligner and light cure through the aligner for 10-20 seconds (depending on the intensity of curing light).

❖❖❖ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

❖ Corrosive.❖ Dispose of properly. Refer to Safety Data Sheet

🔪 **TEMPERATURE LIMITATION:** Store at room temperature (15-25°C)

Protective eyewear should be worn by the patient and dental staff.

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

WARRANTY: Reliance Orthodontic Products Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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