**RELIANCE ORTHODONTIC PRODUCTS, INC.**

1540 W. Thorndale Avenue,

Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

## RelianceOrthodontics.com

**ITC TRANSLATIONS**

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

**EMERGO EUROPE**

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

**AUSTRALIAN SPONSOR**

**EMERGO AUSTRALIA**

Level 20, Tower 1l, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

**U.K. PERSON RESPONSIBLE**

**EMERGO CONSULTING (UK) LIMITED**

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

England, United Kingdom

**SWITZERLAND AUTHORIZED REPRESENTATIVE**

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

# BONDING RESINS WITH FLUORIDE - BONDING SYSTEM

# INSTRUCTIONS FOR USE

# CE 1639 V2 OCT.2023

**EN - ENGLISH**

**INDICATIONS FOR USE:** Reliance Bonding Resins are indicated for use in direct and indirect bonding of orthodontic brackets and appliances.

**INSTRUCTIONS FOR USE:**

- Prepare the tooth surface (Prophy, etch, rinse, dry)
- Mixing: Dispense equal amounts of Bonding Resin Part A&B onto a mixing pad. Using a brush or sponge pellet, mix the two resins thoroughly (5-10 seconds)

3. Application: Apply a thin coat of mixture onto the etched surface. The working time of the Bonding Resin is 40 seconds when measured from the start of mixing. It is not necessary to wait for the bonding resin to set before proceeding to the application of the adhesive.

**SPECIAL SAFETY NOTICE:** Brush applicators are single use devices. It is required that a new brush applicator is used for each application.

☞ Flammable

☞ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

Protective eyewear should be worn by the patient and dental staff.

☞ **TEMPERATURE LIMITATION:** Store at room temperature (15–25°C).

**PATIENT POPULATION:** The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

**CONTRAINDICATIONS:** Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

**SAFE DISPOSAL:** Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**RX ONLY:** Federal law restricts this device to sale or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

**TO REORDER:** #S7AF, #S7B

**BG - BULGARIAN**

**ПОКАЗАНИЯ ЗА УПОТРЕБА:** Assure® Plus All Surface Bonding Resin is intended to increase adhesion to normal, atypical (wet or dry) enamel surfaces, amalgam, gold, stainless steel, porcelain, zirconia, composite restorations, acrylic temporary/pontic teeth.

**INSTRUCTIONS FOR USE:**

- Prepare the tooth surface (Prophy, etch, rinse, dry)
- Mixing: Dispense equal amounts of Bonding Resin Part A&B onto a mixing pad. Using a brush or sponge pellet, mix the two resins thoroughly (5-10 seconds)

3. Application: Apply a thin coat of mixture onto the etched surface. The working time of the Bonding Resin is 40 seconds when measured from the start of mixing. It is not necessary to wait for the bonding resin to set before proceeding to the application of the adhesive.

**SPECIAL SAFETY NOTICE:** Brush applicators are single use devices. It is required that a new brush applicator is used for each application.

☞ Flammable

☞ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

Protective eyewear should be worn by the patient and dental staff.

☞ **TEMPERATURE LIMITATION:** Store at room temperature (15–25°C).

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**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the

responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**RX ONLY:** Federal law restricts this device to sale or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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