



ACCESS™

Cheek Retractor

V10 MAR.2020

INDICATIONS FOR USE: The **ACCESS™** Cheek Retractors are intended for use in pulling the cheeks back from the buccal and labial surfaces of the teeth during orthodontic procedures.

CLEANING INSTRUCTIONS: Can be autoclaved or dry heat sterilized.

IMPORTANT!! When sterilized using a dry heat sterilizer, such as a Cox or Dentronix unit, **DO NOT PLACE ANY OTHER INSTRUMENTS ON TOP OF THE ACCESS™ CHEEK RETRACTOR OR CHEEK RETRACTOR PARTS!** Load other instruments in the tray **FIRST**, then place the **ACCESS™ CHEEK RETRACTOR ON TOP** of the other instruments. When heat sterilized following these guidelines, the **ACCESS™** Cheek Retractor will provide you with an optimum number of cycles up to 350° Fahrenheit.

INSTRUCTIONS FOR USE:

To Adjust Tension: The **ACCESS™** Cheek Retractor comes with tension adjusted for most situations. A “straight” bow is normal and should be adequate for majority of patients when inserted into the mouth.

To Reduce Tension: Remove the cheek retractors from the bow and gently bend the stainless-steel bow to bring the two plastic ends of the bow closer together. This may have to be repeated to hold the adjustment to get a “continuous curve” to the stainless-steel bow. The opposite will increase tension. The cheek retractor is then ready to insert into the mouth.

NOTE: If after several uses and sterilization cycles you observe any discoloration on the retractor, use “SCOTCH BRITE® – Never Scratch™ Wool Soap Pads” to gently and quickly remove the spots.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.



RELIANCE

ORTHODONTIC
PRODUCTS

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.

1-800-323-4348 • www.Relianceorthodontics.com



EC

REP

EMERGO EUROPE

Prinsessegracht 20
2514 AP The Hague
The Netherlands

Australian Sponsor: Emergo Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia

Scan Barcode for additional information

Check out our new learning
platform Reliance U!
www.RelianceU.tech

MD

Made in the USA