RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440
Palm Beach Gardens, FL 33408

REP

MD

EMERGO EUROPE

Westervoortsedijk 60
6827 AT Arnhem

The Netherlands

AUSTRALIAN SPONSOR EMERGO AUSTRALIA

Level 20, Tower 1l, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K PERSON RESPONSIBLE EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge

CB24 9BZ

England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV
Leidschendam-Voorburg, Zug Branch Office
Gotthardstrasse 28, 6302 Zug, Switzerland

ACCESS™ CHEEK RETRACTORS

ACCESS™ CHEEK RETRACTORS

INSTRUCTIONS FOR USE

INSTRUCTIONS FOR USE

RELIANCE® ORTHODONTIC PRODUCTS

SCAN FOR MORE INFORMATION

EN - ENGLISH

INDICATIONS FOR USE: The ACCESS™ Cheek Retractors are intended for use in pulling the cheeks back from the buccal and labial surfaces of the teeth during orthodontic procedures.

CLEANING INSTRUCTIONS: Can be autoclaved or dry heat sterilized.

Important!! When sterilized using a dry heat sterilizer, such as a cox or dentronix unit, do not place any other instruments on top of the access™ cheek retractor or cheek retractor parts! Load other instruments in the tray first, then place the access™ cheek retractor on top of the other instruments. When heat sterilized following these guidelines, the access™ cheek retractor will provide you with an optimum number of cycles up to 350° fahrenheit.

INSTRUCTIONS FOR USE:
TO ADJUST TENSION: The ACCESS™ Cheek Retractor comes with tension adjusted for most situations. A “straight” bow is normal and should be adequate for majority of patients when inserted into the mouth.

TO REDUCE TENSION: Remove the cheek retractors from the bow and gently bend the stainless-steel bow to bring the two plastic ends of the bow closer together. This may have to be repeated to hold the adjustment to get a “continuous curve” to the stainless-steel bow. The opposite will increase tension. The cheek retractor is then ready to insert into the mouth.

NOTE: If after several uses and sterilization cycles you observe any discoloration on the retractor, use “SCOTCH BRITE® – Never Scratch™ Wool Soap Pads” to gently and quickly remove the spots.

Protective eyewear should be worn by the patient and dental staff

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINdicATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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