

# Crown Lok™

## Luting Cement CONTAINS FLUORIDE

**INTENDED USE:** Crown Lok™ is intended for use as a cement for metal crowns and orthodontic bands.

### **INSTRUCTIONS FOR USE:**


#### **ENAMEL PREPARATION**

Prophy the surface with 1st & Final<sup>®</sup> oil-free pumice. Rinse and dry. Apply the etching agent to the tooth. Allow 30 seconds for etching. Rinse and dry thoroughly. With a microetcher, sandblast the inside of the crown, band or bridge.

**BONDING & CEMENTATION:** Mix equal amounts of Crown Lok™ Paste A and Paste B for 15 seconds on the supplied mixing pad and place inside the appliance. The working time from start of mix is 2 minutes. Final set will occur 5 minutes from start of mix.

### **For SELF MIXING DUAL-SYRINGE DELIVERY SYSTEM**

1. Remove cap from the SELF MIXING SYRINGE
2. Place mixing tip by lining up notch in tip hub (A) with the syringe notch (B) and rotating in a clockwise direction until it is in a locked position.

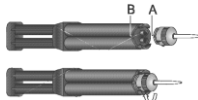

3. Syringe material into band/crown or appliance using mixing tip.  
**Do not re-use the tips. The SELF MIXING SYRINGE mixing tip should be removed and discarded after each patient use. If used intraorally, the syringe should be cold sterilized prior to storage.**
4. Replace the cap on the syringe after use.




**SPECIAL SAFETY NOTE CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES. APPLY DISPOSABLE BARRIER SLEEVES/WRAPPS BEFORE EACH PATIENT**



**PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical

attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

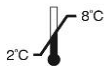

**TEMPERATURE LIMITATION:** Store refrigerated (2-8°C)

**TO REORDER:** Ref# CLSA – Crown Lok Part A Natural Shade, Ref# CLSAB – Crown Lok Part A Blue Shade, Ref# CLSB – Crown Lok Part B, Ref# SMT – Self Mixing Tips, Ref# SMCLB – Self Mixing Crown Lok Blue Syringe, Ref# SMCL – Self Mixing Crown Lok Natural Syringe

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**Rx Only:** Federal law restricts this device to sale by or on the order of a dental professional.



**RELIANCE**<sup>®</sup>

**ORTHODONTIC  
PRODUCTS**

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.  
1-800-323-4348 · [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)



Made in the USA

Scan for more information

Check out our new learning platform Reliance U!  
[www.RelianceU.tech](http://www.RelianceU.tech)

EC

REP

EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Australian Sponsor:  
Emergo Australia  
201 Sussex St. Darling Park  
Tower II, Level 20,  
Sydney, NSW 2000 Australia

MD