

# Enhance™

## ADHESION BOOSTER

### INDICATIONS FOR USE:

Enhance™ is intended for use with any chemical cure, light cure or dual cure adhesive or band cement to increase adhesion to: fluorosed, hypo-calcified, deciduous enamel or dentin; acrylic, composite or metal surfaces; any enamel surface where maximum adhesion is required; enamel that is close to the gingiva where minor surface moisture may reduce bond strength.

### INSTRUCTION FOR USE:

- 1.) Prophyl the enamel surface. Rinse and dry.
  - 2.) Acid etch the enamel for 30 seconds. Rinse and dry thoroughly.
- NOTE: When bonding to a composite, metal or acrylic surface, sandblast the surface, then follow steps 3-6.
- 3.) In mixing well, place a drop of Part A and a drop of Part B Enhance™.
  - 4.) Mix the A&B Enhance™ for 5 seconds and apply 2-3 coats per tooth. Lightly air dry.
  - 5.) Apply the no mix primer, mixed sealant or light cure sealant to the enamel, as usual.
  - 6.) Apply bracket with paste to the tooth.



PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.



Flammable.

②

**Special Safety Note:** Brushes single use devices. It is required that a new brush is used for each patient.



**TEMPERATURE LIMITATION:** Store at room temperature (15-30 Degrees C).

**To Reorder:** Ref# EBAA – 3cc Enhance Part A, Ref# EBAB – 3cc Enhance Part B,  
Ref# EBAA6 – 6cc Enhance Part A, Ref# EBAB6 – 6cc Enhance Part B

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**Rx Only:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.



**RELIANCE**<sup>®</sup>  
ORTHODONTIC  
PRODUCTS

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.  
1-800-323-4348 · [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)

Check out our new  
learning platform Reliance  
U!

[www.RelianceU.tech](http://www.RelianceU.tech)

Made in the USA

EC

REP

EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Australian Sponsor: Emergo  
Australia  
201 Sussex St. Darling Park  
Tower II, Level 20,  
Sydney, NSW 2000 Australia



MD

Scan Barcode for Additional Information