## Scan for more information

## Check out our learning platform Reliance U! www.RelianceU.tech

**Reliance Orthodontic Products, Inc.**  
1540 W. Thorndale Avenue,  
Itasca, IL 60143 USA  
1-800-323-4348 | Made in USA | Rx Only  
RelianceOrthodontics.com

<span><span></span></span>	<span><span></span></span>	<span><span></span></span>	<span><span></span></span>
<span><span></span></span>	<span><span></span></span>	<span><span></span></span>	<span><span></span></span>

**EMERGO EUROPE**  
Prinsessgracht 20, 2514 AP The Hague,  
The Netherlands

**AUSTRALIAN SPONSOR**  
**EMERGO AUSTRALIA**  
Level 20, Tower II, Darling Park, 201 Sussex St,  
Sydney, NSW 2000, Australia

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

## ENHANCE™ ADHESION BOOSTER

## INSTRUCTIONS FOR USE

**V2 MAR.2022**

**RELANCE®**  
**ORTHODONTIC PRODUCTS**

RelianceOrthodontics.com

##### EN - ENGLISH

##### INDICATIONS FOR USE:

Enhance™ is intended for use with any chemical cure, light cure or dual cure adhesive or band cement to increase adhesion to: fluorosed, hypo-calcified, deciduous enamel or dentin; acrylic, composite or metal surfaces; any enamel surface where maximum adhesion is required; enamel that is close to the gingiva where minor surface moisture may reduce bond strength.

##### INSTRUCTION FOR USE:

- Prophy the enamel surface. Rinse and dry.
- Acid etch the enamel for 30 seconds. Rinse and dry thoroughly.
- In mixing well, place a drop of Part A and a drop of Part B Enhance™.
- Mix the A&B Enhance™ for 5 seconds and apply 2-3 coats per tooth. Lightly air dry.
- Apply the no mix primer, mixed sealant or light cure sealant to the enamel, as usual.
- Apply bracket with paste to the tooth.

⚠ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

- ⚠ Flammable.

⚠ **Special Safety Note:** Brushes single use devices. It is required that a new brush is used for each patient.

☞ **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C).

**TO REORDER:** #EBAA, #EBAB, #EBAAG, #EBAAG

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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