## Scan for more information

## Check out our learning platform Reliance U! www.RelianceU.tech

**Reliance Orthodontic Products, Inc.**  
1540 W. Thorndale Avenue,  
Itasca, IL 60143 USA  
1-800-323-4348 | Made in USA | Rx Only  
RelianceOrthodontics.com

**EMERGO EUROPE**  
Prinssessgracht 20, 2514 AP The Hague,  
The Netherlands


**AUSTRALIAN SPONSOR**  
**EMERGO AUSTRALIA**  
Level 20, Tower II, Darling Park, 201 Sussex St,  
Sydney, NSW 2000, Australia

## RELIANCE® ETCHANT

### INSTRUCTIONS FOR USE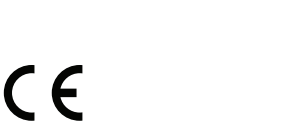

RelianceOrthodontics.com

#### EN - ENGLISH

**INDICATIONS FOR USE:** Reliance Etchants are intended for use on enamel to prepare the surface for bonding.

#### INSTRUCTIONS FOR USE

- Isolate the teeth for etching.
- With a cotton pledget or brush, dab the etching agent onto the entire area to be bonded. Do not rub the etching agent onto the enamel. Allow 30 seconds for etching (60 seconds for deciduous teeth). Avoid placing etching agent on soft tissue.
- Rinse thoroughly (5 seconds per tooth for liquid etchant and 10 seconds per tooth for gel etchant) to stop the etching process and remove demineralized particles.

⚠ **CAUTION:** Corrosive. Etching liquid or gel (phosphoric acid) is a severe eye and tissue irritant. If accidentally instilled into the eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Injury may result if etchant remains on the skin or mucosa for extended periods of time.

⚠⚠ Flammable and Irritant for Cherry only.

🔥 **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C)

**TO REORDER:** #EL24, #EL24C, #EL9, #EG18, #EG9

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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