

Excel®

BONDING ADHESIVE – With and Without Fluoride


INDICATIONS FOR USE: Excel is intended for use as an orthodontic bonding adhesive for brackets and appliances.

BONDING LARGE ACRYLIC APPLIANCES:

1. Before beginning the etching process, apply a thin coat of Reliance Plastic Conditioner to the underside of the acrylic appliance, wait 60 seconds.
2. Prophylax the involved teeth. Etch the buccal and lingual surfaces of teeth that are covered by the appliance. Do not etch the occlusal surface. Rinse and dry.
3. Condition deciduous enamel with Assure® Plus, air dry.
4. Spatulate Excel® Paste part A&B on a mixing pad. A 1 to 1 ratio produces a 3-minute working time for Excel Regular and 60 seconds for Excel Fast.
5. Apply a generous amount of mixed paste to the appliance.
6. Place the appliance into the mouth and clean up flash.
7. Wait 8 minutes before activating the appliance.

APPLICATION TO BRACKET PAD (INDIRECT BONDING):

1. Prophylax rinse and dry,
2. Etch rinse and dry. Apply 1 coat of Assure® Plus, air dry and light cure.
3. Apply a bead of the mixed paste to the bracket base. Slide the tray into the mouth and hold in place for 3 minutes.
4. After 5 minutes, IDB trays can be removed.

 **CAUTION:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water.



TEMPERATURE LIMITATION: Store at room temperature (15-30 °C)

TO REORDER:

ERJA	12 GM EXCEL REGULAR PASTE A-JAR-NON FLUORIDE	1ERSAF	6 GM EXCEL REGULAR PASTE A-SCREW-w/ FLUORIDE
ERJAF	12 GM EXCEL REGULAR PASTE A-JAR- w/FLUORIDE	1ERSB	6 GM EXCEL REGULAR PASTE B-SCREW
ERJB	12 GM EXCEL REGULAR PASTE B-JAR	EFJAF	12 GM EXCEL FAST PASTE A-JAR-w/ FLUORIDE
1ERSA	6 GM EXCEL REGULAR PASTE A-SCREW-NON FLUORIDE	EFJB	12 GM EXCEL FAST PASTE B-JAR
1EFSAF	6 GM EXCEL FAST PASTE A-SCREW-FLUORIDE	1EFSB	6 GM EXCEL FAST PASTE B-SCREW

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Rx Only: Federal law restricts this device to sale by or on the order of a dental professional.



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com

Made in the USA



Australian Sponsor:
Emergo Australia
201 Sussex St.
Darling Park
Tower II, Level 20,
Sydney, NSW 2000
Australia

Check out our new
learning platform
Reliance U!
www.RelianceU.tech