

**Excel®**  
**BONDING ADHESIVE – Contains Fluoride**

**REGULAR SET** (3-minute working time): For the indirect technique, bonding large acrylic appliances and lingual appliances

**INDICATIONS FOR USE:** Excel is intended for use as an orthodontic bonding adhesive for brackets and appliances.

**INSTRUCTIONS FOR USE:**


**BONDING LARGE ACRYLIC APPLIANCES:**


1. Before beginning the etching process, apply a thin coat of Reliance Plastic Conditioner to the underside of the acrylic appliance, wait 60 seconds.
2. Prophyl the involved teeth. Etch the buccal and lingual surfaces of teeth that are covered by the appliance. Do not etch the occlusal surface. Rinse and dry.
3. Dispense equal amounts of Bonding Resin Part A&B and mix thoroughly.  
NOTE– to increase strength or use on deciduous enamel, Assure® Plus should be used in place of bonding resins and air dried. Apply a thin coat of mixture onto the etched surface. The working time is 70 seconds.
4. Spatulate Excel® Paste part A&B on a mixing pad. A 1 to 1 ratio produces a 3-minute working time for Excel Regular.
5. Apply a generous amount of mixed paste to the appliance.
6. Place the appliance into the mouth and clean up flash.
7. Wait 8 minutes before activating the appliance.

**APPLICATION TO BRACKET PAD (INDIRECT BONDING):**

1. Prophyl rinse and dry,
2. Etch rinse and dry.
3. Dispense equal amounts of Bonding Resin Part A&B and mix thoroughly.  
NOTE– to increase strength, Assure® Plus should be used in place of bonding resins and air dried. Apply a thin coat of mixture onto the etched surface. The working time is 70 seconds.
4. Spatulate Excel® Paste part A&B on a mixing pad. A 1 to 1 ratio produces a 3-minute working time for Excel Regular.
5. Apply a bead of the mixed paste to the bracket base. Slide the tray into the mouth and hold in place for 3 minutes.
6. After 5 minutes, IDB trays can be removed.

**Special Safety Note:** Brush applicators, sponge pellets and mixing pads are single use devices. It is required that a new brush applicator and mixing pad sheet are used for each patient. Spatulas and mixing wells can be autoclaved.

 **CAUTION:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water.

 Etching liquids or gels (phosphoric acid) are severe eye and tissue irritants. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Injury may result if etchant is allowed to remain on the skin or mucosa for extended periods of time.



**TEMPERATURE LIMITATION:** Store at room temperature (15-30 °C)

**Contents of Kit:** 12g Catalyst Paste A, 12g Base Paste B, 7g Bonding Resin A, 7g Bonding Resin B, 9g Liquid Etchant, accessory pack.

**TO REORDER:**

EHJF	EXCEL HERBST KIT-JAR-FLUORIDE	1ERSAF	6 GM EXCEL REGULAR PASTE A-SCREW-FLUORIDE
EHSF	EXCEL HERBST KIT-SCREW-FLUORIDE	1ERSB	6 GM EXCEL REGULAR PASTE B-SCREW
EL9	9GM ETCHANT (LIQUID)	ERSF	EXCEL REGULAR KIT SCREW-FLUORIDE
ERJF	EXCEL REGULAR KIT JAR-FLUORIDE	EUERS	EXCEL EU REGULAR KIT SCREW-NONFLUORIDE PASTE WITH FLUORIDE BONDING RESIN
EUERJ	EXCEL EU REGULAR KIT JAR – NONFLUORIDE WITH FLUORIDE BONDING RESINS	PL	10CC PLASTIC CONDITIONER
ERJA	12 GM EXCEL REGULAR PASTE A-JAR-NON FLUORIDE	S20AF	20GM ECONOMY SEALANT RESIN A-FLUORIDE
ERJAF	12 GM EXCEL REGULAR PASTE A-JAR-FLUORIDE	S20B	20GM ECONOMY SEALANT RESIN B
ERJB	12 GM EXCEL REGULAR PASTE B-JAR	S7AF	7GM SEALANT RESIN A-W/ FLUORIDE
1ERSA	6 GM EXCEL REGULAR PASTE A-SCREW-NON FLUORIDE	S7B	7GM SEALANT RESIN B

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**Rx Only:** Federal law restricts this device to sale by or on the order of a dental professional.



**RELIANCE®**

ORTHODONTIC  
PRODUCTS

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.  
1-800-323-4348 · [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands



Made in the USA

Check out our new learning  
platform Reliance U!  
[www.RelianceU.tech](http://www.RelianceU.tech)

Australian Sponsor: Emergo  
Australia  
201 Sussex St.  
Darling Park  
Tower II, Level 20,  
Sydney, NSW 2000 Australia



Scan for more information