RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thornhale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ


England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

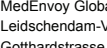EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

EN - ENGLISH

INDICATIONS FOR USE: Reliance Extend LTR is intended for use as a lingual retention wire.

INSTRUCTIONS FOR USE

- Using the provided measuring device, seat the midline notch firmly at the interproximal gap.
- The selection line that fully clears the distal edge of the patient’s lateral wil correspond to one of 5 wire sizes.
- Once the proper size has been selected, a contouring plier can be used for final adaptations. Note: Extend™ wires should be sandblasted on the beveled bonding section of the cuspidis. Uniform flush adaptation across all lingual surfaces is imperative for long-term success.

Chairside intraoral preparation

- Smooth any pronounced marginal ridges on the cuspidis as necessary.
- Prophy and then sandblast the lingual enamel surface on the lower cuspidis. Acid etch for 30 seconds. Rinse and dry thoroughly.
- Apply 1 coat of Assure Plus to the etched tooth surfaces. Lightly air dry.
- Place 3 loops of dental floss in the interproximal contacts at the midline and at the lateral/cuspid contacts.
- Using LCR light cure paste, extrude a small amount of paste over the wire on each cuspid. Use enough paste to cover the wire and bond to the enamel.
- Dip a Microbrush into Assure Plus and spread the paste to a smooth finish in a gingival-incisal motion. Lightly air dry.
- Light cure the paste on each tooth to ensure full polymerization.
- Remove the dental floss and check for any excess composite in the contacts.

⊕ Dispose of Property

⊕ Flammable as a dust

⊗ Do not Re-Use

Protective eyewear should be worn by the patient and dental staff

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.
SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.
RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.
NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

TO REORDER: #EXK, #EX18, #EX20, #EX22, #EX24, #EX26, #EXMD

🇬🇧
INDICATIONS FOR USE: Reliance Extend LTR is intended for use as a lingual retention wire.
INSTRUCTIONS FOR USE
1. Using the provided measuring device, seat the midline notch firmly at the interproximal gap.
2. The selection line that fully clears the distal edge of the patient’s lateral will correspond to one of 5 wire sizes.
3. Once the proper size has been selected, a contouring plier can be used for final adaptations. Note: Extend™ wires should be sandblasted on the beveled bonding section of the cuspidis. Uniform flush adaptation across all lingual surfaces is imperative for long-term success.
Chairside intraoral preparation
1. Smooth any pronounced marginal ridges on the cuspidis as necessary.
2. Prophy and then sandblast the lingual enamel surface on the lower cuspidis. Acid etch for 30 seconds. Rinse and dry thoroughly.
3. Apply 1 coat of Assure Plus to the etched tooth surfaces. Lightly air dry.
4. Place 3 loops of dental floss in the interproximal contacts at the midline and at the lateral/cuspid contacts.
5. Using LCR light cure paste, extrude a small amount of paste over the wire on each cuspid. Use enough paste to cover the wire and bond to the enamel.
6. Dip a Microbrush into Assure Plus and spread the paste to a smooth finish in a gingival-incisal motion. Lightly air dry.
7. Light cure the paste on each tooth to ensure full polymerization.
8. Remove the dental floss and check for any excess composite in the contacts.
⊕ Dispose of Property
⊕ Flammable as a dust
⊗ Do not Re-Use
Protective eyewear should be worn by the patient and dental staff
PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.
CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage. SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection
WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith. RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional. NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.
TO REORDER: #EXK, #EX18, #EX20, #EX22, #EX24, #EX26, #EXMD

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