RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney,

NSW 2000, Australia

U.K. PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

U.K. PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

U.K. PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

U.K. PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

England, United Kingdom

EN - ENGLISH

INDICATIONS FOR USE: Reliance Gel Etchant is 37% phosphoric acid (H3PO4) polymer-thickened and intended for use on enamel to prepare the surface for bonding.

INSTRUCTIONS FOR USE

- Unscrew and remove the white storage cap from the tip of the syringe.
- Snap the needle tip dispenser in place and twist clockwise until needle is locked.
- Isolate the teeth for etching. Apply etching agent onto the entire area to be bonded. Allow 30 seconds for etching.
 - ⚠ CAUTION: Avoid placing etching agent on soft tissue.
- Rinse thoroughly and air dry
- When procedure is completed, remove needle tip, discard needle tip and replace the white storage cap.

⚠ **CAUTION:** Etching gels (contain phosphoric acid) are severe eye and tissue irritants. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Injury may result if etchant is allowed to remain on the skin or mucosa for extended periods of time.

CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES. APPLY DISPOSABLE BARRIER SLEEVES/WRAPLS BEFORE EACH PATIENT. TIPS ARE SINGLE USE DEVICES. A NEW TIP IS REQUIRED FOR EACH PATIENT.

👉 **TEMPERATURE LIMITATION:** Store at room temperature (15-25°C)

Protective eyewear should be worn by the patient and dental staff.

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.
SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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