RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower 1l, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

GLASSLOK™ DUAL CURE

EN - ENGLISH

INDICATIONS FOR USE: GlassLok is intended for use as a dual cure glass ionomer band cement for use in an orthodontic or dental office.

Performance Indicator: Bonds to enamel and metal for duration of treatment.

INSTRUCTIONS FOR USE:

BAND AND APPLIANCE CEMENTATION

- Perform a thorough prophylaxis of the enamel surface to be bonded with 1st & Final® pumice. Rinse and dry thoroughly.
- Dispense 2 Level Scoops of Powder and 4 drops of liquid onto a mixing pad and spatulate thoroughly. Place mixed solution to inside of bands.
- Proceed with placement of bands and clean up flash. Working time is 4 minutes. Chemical cure final set will be achieved in 10 minutes from start of mix. Note: Light curing is suggested to seal the periphery of band. To fully polymerize GlassLok with a curing light, light cure for 20 seconds around the periphery of band.

⚡⚡ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

⚡ Corrosive

⚠ Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

Ⓢ Spatulas, Measuring Scoops and Mixing Pads are single use devices. Do not reuse.

⚡ **TEMPERATURE LIMITATION:** Store at room temperature (15-25°C).

Protective eyewear should be worn by the patient and dental staff

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith. **RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

TO REORDER: #GLL15, #GLL15B, #GLL25, #GLL25B, #GLP15, #GLP100, #GLE, #GLS, #GLEB, #GLSB

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