RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440 Palm Beach Gardens, FL 33408

EMERGO EUROPE

Westervoortsedijk 60 6827 AT Arnhem The Netherlands

AUSTRALIAN SPONSOR EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International Compass House, Vision Park Histon Cambridge CB24 9BZ England, United Kingdom


SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV Leidschendam-Voorburg, Zug Branch Office Gotthardstrasse 28, 6302 Zug, Switzerland

KOBAYASHI TIE HOOKS

INSTRUCTIONS FOR USE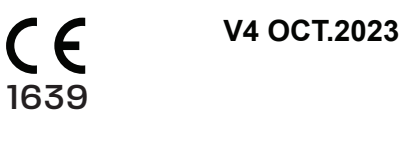

RELIANCE®

ORTHODONTIC PRODUCTS

SCAN FOR MORE INFORMATION

EN - ENGLISH

INDICATIONS FOR USE: Reliance Kobayashi Tie Hooks are intended for use in ligation of archwires and hooks to orthodontic brackets and elastics.

INDICATIONS FOR USE

- Remove single Kobayashi Tie Hook from container. Perform hot or cold sterilization.
- Position open end of the tie hook over the bracket tie wing.
- Twist the two ends (approximately 4 times) until it is firmly seated against the bracket base.
- Cut excess wire off close to the base without compromising the twist.
- Bend the loop to the desired angulation.
- Attach elastic, if desired.

🌀 **Single Use Device**

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the pro duct for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

⚠️🌀 **WARNING:** This product can expose you to Nickel (metallic), which is known to the State of California to cause cancer. For more information, go to www.P65Warnings.ca.gov.

🌀 **Flammable as dust** 🌀 **Dispose of Properly**
PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

Protective eyewear should be worn by the patient and dental staff

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