



KOBAYASHI TIE HOOKS

V9 MAR.2020

INDICATIONS FOR USE: Reliance Kobayashi Tie Hooks are intended for use in ligation of archwires and hooks to orthodontic brackets and elastics.

INSTRUCTION FOR USE

1. Remove single Kobayashi Tie Hook from container. Perform hot or cold sterilization.
2. Position open end of the tie hook over the bracket tie wing.
3. Twist the two ends (approximately 4 times) until it is firmly seated against the bracket base.
4. Cut excess wire off close to the base without compromising the twist.
5. Bend the loop to the desired angulation.
6. Attach elastic, if desired.

Made in the USA

Single Use Device



WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.



WARNING: This product can expose you to Nickel (metallic), which is known to the State of California to cause cancer. For more information, go to www.P65Warnings.ca.gov.



Flammable as dust



Dispose of Properly



Scan for more information



1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com

Australian Sponsor: Emergo Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

MD

Check out our new
learning platform
Reliance U!
www.RelianceU.tech