RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

🇺🇸

🇬🇧

EN - ENGLISH

INDICATIONS FOR USE: L.C.R.™ is a flexible, light cure orthodontic adhesive intended to be used within an orthodontic, dental or pediatric dental office for the bonding of lingual retainers, creation of occlusal buildups and for retention of thermoplastic aligners.

INSTRUCTIONS FOR USE:

LINGUAL RETENTION

1. Prophly the enamel, rinse and dry. Sandblast the enamel, rinse and dry. Acid etch the enamel for 30 seconds. Rinse thoroughly and dry.

2. Dispense a drop or two of Light Bond sealant onto a mixing pad. Apply a thin uniform layer of sealant to the etched enamel. **NOTE**– to increase strength or use on deciduous enamel, Assure® Plus should be used in place of Light Bond Sealants and air dried.

3. Dispense L.C.R.™ over the wire. Shape and smooth paste as desired. Cure the paste on each tooth from the lingual for 30 seconds. **NOTE:** To prevent L.C.R.™ from sticking to an instrument, wet the instrument with Assure® Plus.

NOTE: L.C.R. will fluoresce under the presence of a blacklight.

THERMOPLASTIC ALIGNERS: Prophly the enamel, rinse and dry. Sandblast the enamel, rinse and dry. Acid etch the enamel for 30 seconds. Rinse thoroughly and dry. Condition the prepared enamel with Assure® Plus and air dry. Apply the L.C.R.™ paste into the aligner matrix, deliver and light cure for 30 seconds.

OCCLUSAL BUILD-UP: After proper preparation of the occlusal tooth surface (prophly, etch, rinse, dry), apply the L.C.R.™ to the occlusal surface to the determined height. Light cure 30 seconds.

CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES. APPLY DISPOSABLE BARRIER SLEEVES/WRAPPS BEFORE EACH PATIENT.

Ⓢ **LCR tips are single use devices. A new tip should be used for each patient.**

IMPORTANT: Curing times depend on the intensity of the curing light. The total curing time can range from 10 seconds to 30 seconds. Please refer to the light manufacturer for recommended curing times. Ⓢ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water.

Δ Use of UV, halogen and L.E.D. curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

Protective eyewear should be worn by the patient and dental staff.
~“ TEMPERATURE LIMITATION: Store at room temperature (15-25°C)

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous-waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith. **RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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