**RELIANCE ORTHODONTIC PRODUCTS, INC.**

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

## RelianceOrthodontics.com

**ITC TRANSLATIONS**

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

**EMERGO EUROPE**

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

**AUSTRALIAN SPONSOR**

**EMERGO AUSTRALIA**

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

**U.K. PERSON RESPONSIBLE**

**EMERGO CONSULTING (UK) LIMITED**

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

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
England, United Kingdom

**SWITZERLAND AUTHORIZED REPRESENTATIVE**


MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

## INSTRUCTIONS FOR USE

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SCAN FOR MORE INFORMATION

**EN - ENGLISH**

**INDICATIONS FOR USE:** Reliance Metal Primer is intended for use as a conditioner of metal surfaces for bonding with an adhesive.

**INSTRUCTIONS FOR USE:**

- Prophy and sandblast the metal surface. Rinse and dry thoroughly.
- Apply 1 coat of Reliance Metal Primer on the metal surface area to be bonded. Wait 30 seconds.
- Paint a thin coat of sealant on the conditioned metal surface. Apply the paste to the bracket base and position on the tooth. Light cure

**Special Safety Notice:** This device Brush applicators are single use devices. It is required that a new brush applicator is used for each application.

◆ **Flammable**

◆ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

☞ **TEMPERATURE LIMITATION:** Store at room temperature (15-25°C).

Protective eyewear should be worn by the patient and dental staff.

**PATIENT POPULATION:** The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

**CONTRAINDICATIONS:** Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

**SAFE DISPOSAL:** Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontics Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from use of or suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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