RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney,

NSW 2000, Australia

U.K. PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International

Compass House, Vision Park Histon

Cambridge

CB24 9BZ

England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

EMERGO

USA

UK

FR

DE

IT

ES

GR

DK

PL

CZ

SE

NO

RU

UA

BE

PT

TR

GR

EN - ENGLISH

INDICATIONS FOR USE: Intended for use as a Lingual Retainer Wire. This product is not indicated for individuals with sensitivity to Nickel.

INSTRUCTIONS FOR USE:

- Prophy and sandblast the enamel, rinse and dry. Etch the enamel for 30 seconds. Rinse and Dry.
- Apply one coat of Assure® Plus to the etched enamel. Lightly air dry.
- Apply a minimal amount of L.C.R.™ GoTo™ or FlowTain™ to the lingual surface of each tooth.
 - Place the Ortho FlexTech® chain in the applied adhesive (“undercoat”) on the lingual surface. Position for best alignment and take up any slack in the chain.
 - Alternate Method: Place one end of Ortho FlexTech®chain in the applied adhesive paste (“undercoat”) on the lingual cuspid surface. Position for best alignment and light cure to hold chain in place. Proceed to the next tooth, taking up any slack in the chain. Repeat this process until the opposing cuspid has been reached.

- Place an “overcoat” of L.C.R.™ GoTo™ or FlowTain™ paste on top of the cured material to cover the Ortho FlexTech™ chain. Light cure the “overcoat” paste on each bonded tooth.

- Check for interference between upper and lower. Polish any rough spots or high points carefully.

NOTE: If used on upper to hold diastema, consider secondary .0175 twist 1 to 1 or Retainium™, as the chain can stretch slightly allowing space to reopen.

Ⓢ Do not Re-Use. This is a single use device Protective eyewear should be worn by the patient and dental staff.

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontics Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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