**RELIANCE ORTHODONTIC PRODUCTS, INC.**

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only  
RelianceOrthodontics.com

## RelianceOrthodontics.com

**ITC TRANSLATIONS**  
ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

**EMERGO EUROPE**

Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

**AUSTRALIAN SPONSOR**

**EMERGO AUSTRALIA**

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

**U.K. PERSON RESPONSIBLE**

**EMERGO CONSULTING (UK) LIMITED**

c/o Cr360 – UL International  
Compass House, Vision Park Histon  
Cambridge  
CB24 9BZ  
England, United Kingdom

**SWITZERLAND AUTHORIZED REPRESENTATIVE**

MedEnvoy Global BV  
Leidschendam-Voorburg, Zug Branch Office  
Gotthardstrasse 28, 6302 Zug, Switzerland

# PRO SEAL® & LED PRO SEAL®

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

**TO REORDER:** #LEDPRO, #PRO

## INSTRUCTIONS FOR USE

1639

**ORTHODONTIC PRODUCTS**

**EN - ENGLISH**

**INDICATIONS FOR USE:** PRO SEAL® and LED PRO SEAL® are intended for use as an enamel surface sealant around orthodontic brackets.

**INSTRUCTIONS FOR USE:**

**PROPHYLAXIS:** Prophly the enamel surface with 1st & Final® or plain pumice. Rinse and dry thoroughly.

**ETCH:** Apply etching agent onto the entire area to be bonded. Allow 30 seconds for etching (60 for deciduous teeth). Rinse and dry thoroughly. NOTE: Atypical enamel should be first etched, conditioned with one coat of Assure® Plus then lightly dried before the PRO SEAL® is applied.

**APPLICATION OF PRO SEAL®:** With a bristle brush (do not use a Microbrush), apply a thin, uniform layer to the etched enamel. Stroke over with the same brush to ensure a thin layer and proper coverage. Cure each tooth for 6-10 seconds with a curing light.

**IMPORTANT:** PRO SEAL® must be cured with a halogen or a broad-wave LED light with high intensity in the 390-440 nM wavelength range. LED PRO SEAL® can be cured with any curing light.

**REMOVAL OF SEALANT RESIN:** Lightly polish the entire tooth surface with a Renew Finishing #383 rubber point where PRO SEAL® and LED PRO SEAL® has been applied.

❖❖ **CAUTION:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water.

Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

❖ Corrosive      ❖ Dispose of property. Harmful to aquatic life.

~❄️ **TEMPERATURE LIMITATION:** Store at room temperature (15-25°C)

Protective eyewear should be worn by the patient and dental staff

**PATIENT POPULATION:** The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

**CONTRAINDICATIONS:** Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

**SAFE DISPOSAL:** Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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