

Pad Lock®

FLUORESCENT LIGHT CURE BRACKET BONDING SYSTEM WITH OR WITHOUT FLUORIDE

INDICATIONS FOR USE: Pad Lock® is intended for use as a fluorescing, light cure orthodontic bracket adhesive.

INSTRUCTIONS FOR USE:

Tooth Preparation: Prophyl the enamel surface. Rinse and dry. Etch the enamel surface. Rinse and dry. Apply 1 coat of Light Bond Sealant or Assure® Plus and air dry.

Bracket Application: Using a spatula, smear **Pad Lock®** paste into the mesh of the bracket pad. Place bracket on the tooth and press in the desired position. Remove flash and light cure.



PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.



Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.



Etchants are corrosive.

Special Safety Note: Brush applicators and mixing pads are single use devices. It is required that a new brush applicator and mixing pad sheet are used for each patient. Mixing wells can be autoclaved.



TEMPERATURE LIMITATION: Store refrigerated (2-8 degrees C). Avoid excessive heat. Allow product to reach room temperature prior to use.

CONTENTS OF KIT: (4) 4.5g Paste in push syringes or tips, (2) 3cc Light Bond™ sealants, 9g Liquid Etchant, accessory pack.

TO REORDER:

PLP	4.5GM PAD LOCK PASTE-PUSH-NON-FLUORIDE	PLPKFAS	PAD LOCK ECON KIT-PUSH-W/FLUORIDE & ASSURE SEALANT	PLPKAS	PAD LOCK ECON KIT-PUSH-NON-FLUORIDE & ASSURE SEALANT
PLPF	4.5GM PAD LOCK PASTE-PUSH W/FLUORIDE	PLT	4.5GM PAD LOCK PASTE IN TIPS- NON-FLUORIDE	PLTP	4.5 GM PAD LOCK THIN IN PUSH SYRINGE - NON FLUORIDE
PLPKF	PAD LOCK ECONOMY KIT PUSH W/FLUORIDE	PLTF	4.5GM PAD LOCK PASTE IN TIPS W/FLUORIDE		

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

Rx Only: Federal law restricts this device to sale or on the order of a dental professional.



RELIANCE®

ORTHODONTIC
PRODUCTS

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 · www.RelianceOrthodontics.com

Australian Sponsor:
Emergo Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia

Check out our new learning
platform Reliance UI
www.RelianceU.tech



Made in the USA



EC

REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Scan for more information