Scan for more information

Check out our learning platform Reliance U! **www.RelianceU.tech**

Reliance Orthodontic Products, Inc.
1540 W. Thorndale Avenue,
Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

EC **REP** **MD**

EMERGO EUROPE
Prinssgracht 20, 2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: Federal law restricts this device to sale by or on the order of a dental professional.
NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

PHASE II® CHEMICAL CURE ADHESIVE

INSTRUCTIONS FOR USE

CE **2797** **V2 MAR.2022**

RELIANCE®
ORTHODONTIC PRODUCTS

RelianceOrthodontics.com

EN - ENGLISH
INDICATIONS FOR USE: Phase II is intended for use in direct and indirect bonding of orthodontic brackets and appliances.
INSTRUCTIONS FOR USE:
PROPHYLAXIS: Prophy the tooth with 1st & Final® pumice. Rinse thoroughly and dry.
ETCH: Apply the etching agent onto the entire area to be bonded. Allow 30 seconds for etching. Rinse and dry thoroughly.
MIXING & APPLICATION OF BONDING RESIN/PASTE:
1.Apply 1 coat of Assure® Plus to the etched surface, air dry
2.Dispense equal amounts of Paste A&B onto a mixing pad and spatulate thoroughly.
APPLICATION TO BRACKET PAD:
Apply mixed paste to bracket pad, place on tooth and press to express flash. The working time is 2.5 minutes (DCJAF/1DCSAF: 4 Minutes). Active archwires may be inserted after 5 minutes (DCJAF/1DCSAF: 7 Minutes).
PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water.

TEMPERATURE LIMITATION: Store at room temperature (15-30°C).
REORDER INFORMATION: #PJAF, #PJB, #1PSAF, #1PSB

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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- Nanesite 1 sloj proizvoda Assure © Plus na jetkanu površinu, osušite zrakom