

Phase II®

CHEMICAL CURE ADHESIVE WITH OR WITHOUT FLUORIDE BONDING SYSTEM FOR DIRECT AND INDIRECT BONDING

INDICATIONS FOR USE: Phase II is intended for use in direct and indirect bonding of orthodontic brackets and appliances.

INSTRUCTIONS FOR USE:

PROPHYLAXIS: Prophylax the tooth with 1st & Final® pumice. Rinse thoroughly and dry.

ETCH: Apply the etching agent onto the entire area to be bonded. Allow 30 seconds for etching. Rinse and dry thoroughly.

MIXING & APPLICATION OF BONDING RESIN/PASTE:

- 1) Apply 1 coat of Assure® Plus to the etched surface, air dry
- 2) Dispense equal amounts of Paste A&B onto a mixing pad and spatulate thoroughly.

APPLICATION TO BRACKET PAD:

Apply mixed paste to bracket pad, place on tooth and press to express flash. The working time is 2.5-minutes. Active archwires may be inserted after 5 minutes.



PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water.



TEMPERATURE LIMITATION: Store at room temperature (15-30°C).

Reorder Information: Components: Ref# PJAF: Phase II Part A w/Fluoride in Jar, Ref# PJB: Phase II Part B in Jar, Ref# 1PSAF: Phase II Part A w/Fluoride in Screw Syringe, Ref# 1PSB Phase II Part B in Screw Syringe

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Rx Only: Federal law restricts this device to sale by or on the order of a dental professional.



RELIANCE[®]
ORTHODONTIC
PRODUCTS



1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 · www.RelianceOrthodontics.com



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Check out our new learning
platform Reliance U!
www.RelianceU.tech

Made in the USA

Australian Sponsor:
Emergo Australia
Level 20, Tower II, Darling
Park 201
Sussex St.
Sydney, NSW 2000
Australia



MD

Scan for information