RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

		
EC REP	MD	
 	 	

EMERGO EUROPE

Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE
EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge
CB24 9BZ
England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV
Leidschendam-Voorburg, Zug Branch Office
Gotthardstrasse 28, 6302 Zug, Switzerland


PORCELAIN CONDITIONER

INSTRUCTIONS FOR USE

INSTRUCTIONS FOR USE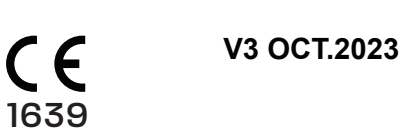

RELIANCE[®]
ORTHODONTIC
PRODUCTS

SCAN FOR MORE INFORMATION

EN - ENGLISH

INTENDED USE: Reliance Porcelain Conditioner is intended for use in the preparation of glazed dental porcelain for bonding. Porcelain Conditioner is a silane coupling agent which chemically unites with the silicon in the porcelain and the acrylic in chemical, light cure and dual cure adhesives.

INSTRUCTIONS FOR USE:

- Sandblast the porcelain surface. Rinse and dry.
- Apply one THIN coat of Reliance Porcelain Conditioner. Isolate and wait 2 full minutes. Thoroughly dry with air. If a warm air tooth dryer is utilized to dry, the wait time can be reduced to 1 minute.
- Apply 1 coat of Assure Plus, lightly dry with air and light cure for 10 seconds.
- Proceed with bracket placement.

⚠ **PRECAUTIONS:** May cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse area immediately with copious amounts of water for several minutes.

CAUTION: DO NOT REUSE IF BOTTLE BECOMES CONTAMINATED. A NEW APPLICATOR IS REQUIRED FOR EACH PATIENT TO AVOID CONTAMINATION RISK.

⚠ Porcelain Conditioner is flammable.

⚠ **TEMPERATURE LIMITATION:** Store at room temperature (15-25°C)

Protective eyewear should be worn by the patient and dental staff

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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