

Rely·a·Bond®
NO MIX ORTHODONTIC ADHESIVE
With or Without Fluoride

INDICATIONS FOR USE: Rely·a·Bond® is intended for use in bonding metal, composite and ceramic orthodontic brackets.

CONTRAINICATION: Rely A Bond is not indicated for bonding lingual retainers without bonding pads.

INSTRUCTIONS FOR USE:


PROPHYLAXIS: Prophylax the enamel surface. Rinse and dry.

ETCH: Etch the enamel surface. Rinse and dry.

BONDING PROCEDURE: Apply a thin coat of primer to each etched and dried tooth in the area which the bracket is to be bonded, **and** the underside of each bracket base. With a spatula, apply a small quantity of adhesive paste to the underside of the bracket base and place on tooth. Within 25 seconds, position the bracket to desired angulation and press firmly. Excess adhesive can be easily removed from the periphery of the bracket base after all of the brackets are in place. An active archwire can be inserted 5 minutes after bracket placement.

PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

PRECAUTIONS: Rely A Bond Primers are corrosive.

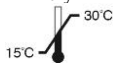

TEMPERATURE LIMITATION: Store at room temperature (15-30 Degrees C).

TO REORDER:

REF # 1RPP 3.5g Rely•a•Bond® Paste in Push Syringe, REF # RPT-14g Rely•a•Bond® Paste in Tips, REF # RPJ-14g Rely•a•Bond® Paste in Jars, REF# 1RPPT – Rely A Bond Thin Paste in Push syringe, 3.5gm,

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



Scan for more information



RELIANCE®
ORTHODONTIC
PRODUCTS

Made in the USA

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 · www.RelianceOrthodontics.com

Check out our new
learning platform
Reliance U!
www.RelianceU.tec

Australian Sponsor:
Emergo Australia
201 Sussex St.
Darling Park
Tower II, Level 20,
Sydney, NSW 2000
Australia