

**Rely-a-Bond®**  
**NO MIX ORTHODONTIC BONDING SYSTEM**  
**WITH OR WITHOUT FLUORIDE**

**INDICATIONS FOR USE:** Rely•a•Bond® is intended for use in bonding metal, composite and ceramic orthodontic brackets.


**Contraindication:** A No Mix System is not recommended for bonding lingual retainers without bonding pads.


**INSTRUCTIONS FOR USE:**

**PROPHYLAXIS:** Prophylax the enamel with 1st & Final® pumice. Rinse and dry.

**ETCH:** Apply etchant to the enamel. Allow 30 seconds for etching. Rinse and dry


**BONDING PROCEDURE:** Apply a thin coat of primer to each etched and dried tooth and the underside of each bracket base. With a spatula, apply a small quantity of adhesive paste to the underside of the bracket base and place on tooth. Within 25 seconds, position the bracket to desired angulation and press firmly. Excess adhesive can be easily removed from the periphery of the bracket base after all of the brackets are in place. An active archwire can be inserted 5 minutes after bracket placement.

 **Special Safety Note:** Brush applicators and mixing pads are single use devices. It is required that a new brush applicator and mixing pad sheet are used for each patient. Spatulas and mixing wells can be autoclaved.

 **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.



Etchants and Primer are Corrosive

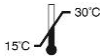

**TEMPERATURE LIMITATION:** Store at room temperature (15-30 Degrees C).  
Protect from excessive heat.

**CONTENTS OF KIT:** 14g Adhesive Paste (jar, syringe tips or push syringes), 16g Primer, 9gm Liquid Etchant, Accessory Pack.

**TO REORDER:**

1RPP	3.5 GM RELY A BOND PASTE - PUSH	1RPPT	3.5 GM RELY A BOND THIN PASTE - PUSH
RJ	RELY A BOND KIT - JAR –NON-FLUORIDE	RPR	16GM RELY A BOND PRIMER NONFLUORIDE
RJF	RELY A BOND KIT - JAR - FLUORIDE	RPRF	16GM RELY A BOND PRIMER - W/FLUORIDE
RP	RELY A BOND KIT - PUSH NONFLUORIDE	RPT	14GM RELY A BOND PASTE - TIPS
RPF	RELY A BOND KIT - PUSH - FLUORIDE	RT	RELY A BOND KIT - TIPS –NON-FLUORIDE
RPJ	14GM RELY A BOND PASTE - JAR	RTF	RELY A BOND KIT - TIPS - FLUORIDE
EL9	9GM ETCHANT (LIQUID)		

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**Rx Only:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.



**RELIANCE**<sup>®</sup>  
ORTHODONTIC  
PRODUCTS

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.  
1-800-323-4348 • [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)



Scan for more information



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Australian Sponsor:  
Emergo Australia  
201 Sussex St.  
Darling Park  
Tower II, Level 20,  
Sydney, NSW 2000  
Australia

MD

Check out our new learning  
platform Reliance U!  
[www.RelianceU.tech](http://www.RelianceU.tech)

Made in the USA