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**EN - ENGLISH**

**INDICATIONS FOR USE:** The Renew® Finishing System is intended for use in the removal of composite paste after debonding and polishing of the surface of enamel, porcelain, metal or composite.

**INSTRUCTION FOR USE:**

The Renew® System Burs included in the Renew® Finishing System are:

- #118L (Ref# RSLB) 18 fluted carbide bur

- #118S (Ref# RSB) 18 fluted carbide bur with shorter shank and cutting surface

- #218 (Ref# RSB2) 18 fluted carbide bur with a tapered cutting surface

- #815 (Ref# RSB815) 12 fluted carbide bur with a round design for posterior work

- #129 (Ref# DB) Medium friction Diamond bur used to roughen composite for bonding

**CAUTION:** Renew® System Burs are friction grip, low speed with wide flutes to help self-clean. These burs should run at approximately 20,000 RPM. The Diamond bur should run between 10,000-20,000 RPM. The operator may use the hand piece with water spray, but it is not necessary.

**SAFETY IN USE:**

- Eye protection must be worn to protect against ejected particles.

- Surgical mask must be worn to avoid inhalation of aerosol or dust generated.

- Inspect the bur for broken flutes before each use and discard defective bur.

- Ensure that the bur is fully seated and gripped in the handpiece collet.

- Maintain handpieces in good working order and correctly lubricated.

- Do not exceed the maximum speed identified above in Caution section.

- Move the bur continuously when in use to avoid localized heating.

**PATIENT POPULATION:** The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

® Renew System Burs are single use devices.
**Δ WARNING!** Used burs should be considered as contaminated and appropriate handling precautions should be taken during reprocessing. Gloves, eye protection and a mask should be worn. Other measures may be required if there are specific infection or cross-contamination risks from the patient.

Protective eyewear should be worn by the patient and dental staff.
**LIMITATIONS OF PROCESSING:** The end of life of the carbide burs are determined by wear and damage in use and the burs should be inspected for defects before use.

**PATIENT POPULATION:** The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

**CONTRAINDICATIONS:** Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

**SAFE DISPOSAL:** Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous-waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance orthodontics Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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© Renew System brusilice su namijenjene za jednokratnu upotrebu.