

Renew[®] Finishing System Points

INDICATIONS FOR USE: The Renew[®] Finishing System Points are intended for use in the removal of thin layers of composite and polishing of enamel or artificial substrates.

INSTRUCTION FOR USE:

1. Ensure that the point is fully seated and gripped in the handpiece collet before use.
2. **IMPORTANT! Do not exceed the maximum speed of 20,000 RPM.** Find the setting for your handpiece that will ensure a speed not to exceed 20,000 RPM and adjust your air pressure accordingly.

SAFETY IN USE:

- Eye protection must be worn to protect against ejected particles.
- Surgical mask must be worn to avoid inhalation of aerosol or dust generated.
- Move the point continuously when in use to avoid localized heating.



Single Use Device



WARNING!! Used polishing points should be considered as contaminated and appropriate handling precautions should be taken. Gloves, eye protection and a mask should be worn. Other measures may be required if there are specific infection or cross-contamination risks from the patient.

LIMITATIONS OF PROCESSING: The end of life of the polishing points are determined by wear and damage in use and the points should be inspected for defects during the preliminary cleaning process.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

REORDER INFORMATION: Friction Grip (REF# RSPFG), Handpiece (REF# RSPHP), Latch (REF# RSPL)

MD



RELIANCE[®]

ORTHODONTIC
PRODUCTS



1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com

Check out our new learning
platform Reliance U!
www.RelianceU.tech



EC

REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Australian Sponsor: Emergo
Australia
201 Sussex St.
Darling Park, Tower II, Level
20, Sydney, NSW 2000
Australia

Scan for additional information

Made in the USA