RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

📖

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440
Palm Beach Gardens, FL 33408

EC REP **MD**

EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

AUSTRALIAN SPONSOR EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE EMERGO CONSULTING (UK) LIMITED
c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge
CB24 9BZ
England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE
MedEnvoy Global BV
Leidschendam-Voorburg, Zug Branch Office
Gotthardstrasse 28, 6302 Zug, Switzerland

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🇬🇧 **RELIANCE®**
ORTHODONTIC PRODUCTS

EN - ENGLISH

INDICATIONS FOR USE: The Renew® Finishing System Points are intended for use in the removal of thin layers of composite and polishing of enamel or artificial substrates.

INSTRUCTION FOR USE:

1.Ensure that the point is fully seated and gripped in the handpiece collet before use.

2.IMPORTANT! Do not exceed the maximum speed of 20,000 RPM. Find the setting for your handpiece that will ensure a speed not to exceed 20,000 RPM and adjust your air pressure accordingly.

SAFETY IN USE:

- Eye protection must be worn to protect against ejected particles.
- Surgical mask must be worn to avoid inhalation of aerosol or dust generated.
- Move the point continuously when in use to avoid localized heating.

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

© Single Use Device

⚠ **WARNING!! Used polishing points should be considered as contaminated and appropriate handling precautions should be taken. Gloves, eye protection and a mask should be worn. Other measures may be required if there are specific infection or cross-contamination risks from the patient.**

Protective eyewear should be worn by the patient and dental staff

LIMITATIONS OF PROCESSING: The end of life of the polishing points are determined by wear and damage in use and the points should be inspected for defects before use.

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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