RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

EMERGO EUROPE

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands

AUSTRALIAN SPONSOR

EMERGO AUSTRALIA

Level 20, Tower 1l, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

U.K. PERSON RESPONSIBLE

EMERGO CONSULTING (UK) LIMITED

c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge

CB24 9BZ

England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE

MedEnvoy Global BV

Leidschendam-Voorburg, Zug Branch Office

Gotthardstrasse 28, 6302 Zug, Switzerland

HILGERS BOND-A-BRAID® AND RETAINIUM

INSTRUCTIONS FOR USE

1639

ORTHODONTIC PRODUCTS

SCAN FOR MORE INFORMATION

EN - ENGLISH

INDICATIONS FOR USE: Bond-a-Braid® and Retainium® are intended for use as a lingual retainer wire for retaining tooth position after orthodontic treatment. Retainium can be used on patients with known nickel allergies.

Bond A Braid is not indicated for individuals with nickel allergies. Retainium is not indicated for individuals with known titanium allergies.

INSTRUCTIONS FOR USE:

ENAMEL PREPARATION Prophy the enamel surfaces with 1st & Final® pumice. Rinse and dry. Sandblast the lingual surfaces of the teeth. Etch the enamel surface for 30 seconds. Rinse and dry thoroughly.

APPLICATION OF BONDING RESIN: Apply one coat of Assure Plus® to each prepared tooth and stroke over to ensure penetration. Lightly air dry.

WIRE DELIVERY:

Step 1. Cut a piece of Retainium wire to the desired length.

Step 2. Make a 45° bend in a gingival direction at the end of the splint, approximately 2-3 mm. This bend helps contour the wire and locks it into the adhesive, preventing any potential sliding.

Step 3. Holding the Retainium splint on its inciso-gingival edge with a flat plier, gently sweep a curve gingivally to match that of the lingual surface.

Step 4. Holding the Retainium splint on its flat edge with same flat plier, gently sweep a curve bucco-lingually.

Step 5. Using floss interproximally, hold the Retainium splint against the lingual surface of the teeth (fig 5). Burnish the splint against the teeth to maximize contact and bond into place.

Note: If Bond A Braid or Retainium® is used on the upper 1-1 or 2-2, it is recommended that a clear thermoplastic retainer be incorporated with the Bond A Braid/ Retainium® wire for best results. Bond the retainer wire first, take a snap impression and then construct the thermoplastic retainer.

☹ Do not Re-Use. Single Use device

☹☹ Fumes/dust generated from stainless steel micro abrasion may produce irritation of the eyes and respiratory system

☹☹ Stainless steel dust is flammable.

☹ Protective eyewear should be worn by the patient and dental staff.

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

TO REORDER: #BAB2, #RET, #XLRET

© 2023 Reliance Orthodontic Products, Inc. All rights reserved.

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