



INDICATIONS FOR USE: The Reliance S.E.P.™ is a self etching primer intended to be used for the preparation of an enamel surface prior to bonding with a light cure orthodontic adhesive.

INSTRUCTIONS FOR USE:


TO DISPENSE PRODUCT FOR USE:

1. Prophyl, rinse and dry the enamel.
2. Position both tips over the light impervious mixing well and **push (Button 1 below) one time with a quick, forceful motion**. Initial dispensing of S.E.P.™ may require one or two pushes to start the flow (**Note: refill must reach room temperature in order to dispense properly**). The ratio that is dispensed is 3:1 (yellow:clear) liquids with a Micro Brush®.
3. Mix the dispensed liquids thoroughly.
4. Saturate a Micro Brush® and vigorously scrub the enamel surface for 5 seconds where the bracket will be placed and slightly beyond. Slide the shield over the mixed solution to protect from ambient light. This mixture can be used for up to 6 hours if protected from light with the shield.
5. Dry the enamel surface thoroughly.
6. Optional – Apply 1 coat of Assure® Plus and air dry to significantly increase strength
7. Apply bracket with light cure paste and light cure.
8. Replace cap tightly and leave the cartridge in the dispenser.

TO LOAD A NEW CARTRIDGE:

1. **Allow Refill Cartridge to reach room temperature prior to use.**
2. **SLIDE EJECT BUTTON SEVERAL TIMES TO RETRACT INTERNAL PLUNGERS BEFORE INSERTING NEW CARTRIDGE. (Refer to Figure 2).**
3. **With Reliance logos facing up on both cartridge and dispenser**, line up the arrow on the dispenser and the arrow on the cartridge then insert cartridge into the S.E.P.™ dispenser holding small barrel toward the top and pushing the Reliance logos towards each other to snap into place.

Figure 1

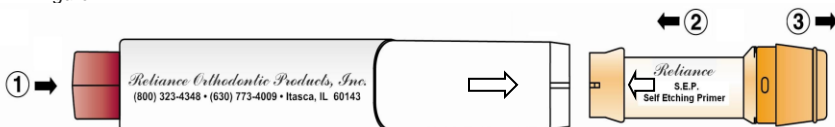

4. Remove orange cap.

TO EJECT THE CARTRIDGE

1. To remove cartridge, turn the dispenser so the Reliance logo is facing down. With cap removed, slide the eject button.
2. Pull the cartridge out of the dispenser.
3. Discard cap and cartridge.
4. **SLIDE EJECT BUTTON SEVERAL TIMES TO RETRACT INTERNAL PLUNGERS BEFORE INSERTING NEW CARTRIDGE.**

Figure 2:



CAUTION: Irritant. Avoid contact with skin or eyes. If accidentally instilled within the eye, flush with copious amounts of water for a minimum of 15 minutes and seek medical attention if irritation persists. If contact occurs with skin or other tissues, wash area with soap and water immediately. Seek medical attention if irritation persists.



Flammable



PRECAUTIONS:

1. Avoid applying excessive amounts of the S.E.P.™ to the enamel surface as the acid contained in the S.E.P.™ can irritate the gingiva (soft tissue) as well as potentially reduce bond strength.
2. Dropping the dispenser will shorten its anticipated life. Dropping with cartridges inserted can also do undetermined damage to the tips. **THE ANTICIPATED LIFE OF THE S.E.P.™ DISPENSER IS ONE YEAR.**
3. MicroBrush applicators are single use devices. It is required that a new brush applicator is used for each application.
4. Removing a partially used cartridge and re-inserting it does cause added stress on the dispenser and cartridge. The cartridge is not intended to be removed until completely used, then to be removed and discarded. If necessary, the cartridge can be removed when only partially used and reinserted for continued use. However, it is important to know that resetting and removing a partially used cartridge requires continued clicking to get back to a point where it can be used again, putting added stress and wear on the internal mechanism and shortening its anticipated life.
5. The dispenser should never be placed in an autoclave or fully submersed in any type of cleaning solution. **It is best to disinfect the dispenser by wiping with a glutaraldehyde solution or denatured alcohol.** Please note that certain surface cleaning agents that contain, but not limited to, substances such as Chlorides, Quaternary Compounds and Citric Acid should be avoided as they may damage dispenser.
6. Do not expose the cartridge or dispenser to areas of high light exposure for any extended time period or the material may cure in the cartridge. The orange tint of the cartridge helps to reduce light penetration to the light sensitive material inside, however it does not eliminate it completely. Caps should also be replaced on the tips of the cartridges immediately after dispensing since the opening at the tip allows unprotected resin to be exposed to ambient light.
7. Mixing wells should be sterilized in a cold sterilization unit, **DO NOT HEAT STERILIZE.**
8. **DO NOT REFRIGERATE DISPENSER.**



TEMPERATURE LIMITATIONS: Store extra refill cartridges refrigerated (2-8 °C). Do not freeze. **Allow product to reach room temperature prior to use.** Avoid exposure to light

TO REORDER: S.E.P Starter Kit – Ref# SEPPRO S.E.P. Refill Cartridge - Ref# SEPR Light Impervious Mixing Well – Ref# SEPW Microbrushes - Ref# MB S.E.P Dispenser - Ref# SEPD
WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Reliance warrants the S.E.P.™ Dispenser for 6 months from the date of purchase with limitations that include misuse or abuse.
Rx Only: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.



RELIANCE®

ORTHODONTIC PRODUCTS

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 · www.RelianceOrthodontics.com



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Australian Sponsor: Emergo
Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia

Made in the USA



Scan for more information

Check out our new learning platform Reliance U!
www.RelianceU.tech