



V10 MAR.2020


# Therma-Cure™


(REF# TC)

**INDICATIONS FOR USE:** Therma-Cure is intended for use as a heat curable adhesive for indirect bonding.

## **INSTRUCTIONS FOR USE:**

1. Condition model with appropriate separating medium.
2. Place a uniform coat of Therma-Cure™ to bracket base and smear into mesh.
3. Place the bracket on the stone model and position. Press to expel excess adhesive. Clean all flash from the periphery of the bracket base.
4. When all brackets are in their final desired position, place the model in a toaster oven for 15 minutes at a temperature setting of 325°F. (Note: The temperature setting of the oven should be checked for accuracy with an oven thermometer.)
5. Remove model and allow to cool. Therma-Cure™ is now fully polymerized and ready for indirect bonding tray fabrication.

 **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

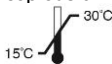

**TEMPERATURE LIMITATION:** Store at room temperature (15-30°C)

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**Rx Only:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.



**RELIANCE**<sup>®</sup>

ORTHODONTIC  
PRODUCTS



1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.  
1-800-323-4348 • [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)



EC

REP

EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

MD

Australian Sponsor: Emergo  
Australia  
201 Sussex St.  
Darling Park, Tower II, Level  
20, Sydney, NSW 2000  
Australia

Check out our new learning  
platform Reliance U!  
[www.RelianceU.tech](http://www.RelianceU.tech)

Made in the USA