RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

EN - ENGLISH

INDICATIONS FOR USE: Therma-Cure is intended for use as a heat curable adhesive for indirect bonding.

INSTRUCTIONS FOR USE:

- Condition model with appropriate separating medium.
- Place a uniform coat of Therma-Cure™ to bracket base and smear into mesh.
- Place the bracket on the stone model and position. Press to expel excess adhesive. Clean all flash from the periphery of the bracket base.
- When all brackets are in their final desired position, place the model in a toaster oven for 15 minutes at a temperature setting of 325°F. (Note: The temperature setting of the oven should be checked for accuracy with an oven thermometer.)
- Remove model and allow to cool. Therma-Cure™ is now fully polymerized and ready for indirect bonding tray fabrication.

⚠ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

⚠ **TEMPERATURE LIMITATION:** Store at room temperature (15-25°C)

Protective eyewear should be worn by the patient and dental staff

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/ or international regulation, a licensed hazardous-waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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