RELIANCE ORTHODONTIC PRODUCTS, INC.
1540 W. Thorndale Avenue,
Itasca, IL 60143 USA
1-800-323-4348 Made in USA | Rx Only
RelianceOrthodontics.com

 RelianceOrthodontics.com

 ITC TRANSLATIONS
ITC USA
2000 PGA Blvd., Suite 4440
Palm Beach Gardens, FL 33408

 EC REP MD

 EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

 AUSTRALIAN SPONSOR
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

 U.K. PERSON RESPONSIBLE
EMERGO CONSULTING (UK) LIMITED
c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge
CB24 9BZ
England, United Kingdom

 SWITZERLAND AUTHORIZED REPRESENTATIVE
MedEnvoy Global BV
Leidschendam-Voorburg, Zug Branch Office
Gotthardstrasse 28, 6302 Zug, Switzerland

ULTRA SEP™ SINGLE BOTTLE SELF ETCHING PRIMER

INSTRUCTIONS FOR USE

V3 OCT.2023

 CE
1639

 RELIANCE®
ORTHODONTIC
PRODUCTS

SCAN FOR MORE INFORMATION

EN - ENGLISH

INDICATIONS FOR USE: UltraSEP is a self etching primer intended to be used for the preparation of an enamel surface prior to bonding with a light cure orthodontic adhesive.

Performance Indicator: Effective bond strength to enamel substrate for duration of treatment.

INSTRUCTIONS FOR USE:

1. Propry, rinse and dry the enamel.
2. Remove the cap from the UltraSEP bottle and dispense 1 drop into a mixing well. One drop will etch 6 teeth.
3. Saturate a microbrush and vigorously scrub the enamel surface for 5 seconds where the bracket will be placed and slightly beyond.
4. Dry the enamel surface for 2-3 seconds
5. Apply the bracket with light cure paste and light cure.
6. Replace the cap on the UltraSEP bottle.

 **PRECAUTIONS:** Flammable, Irritant
Protective eyewear should be worn by the patient and dental staff.

TEMPERATURE LIMITATION:

Store refrigerated. (2-8°C).

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

CONTRAINdications: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous-waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.
NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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